ALLROUNDER

GB

User manual

NL

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D

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N

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S

Bruksanvisning

DK

Brugermanual

SF

Käyttöohje

F

Mode d'emploi

INTRODUCTION

Your Vicair® Academy Allrounder is designed to be used as described in this manual. Please read the whole manual before use.

The Allrounder consists of the following parts:

Outer cover & inner cushion

The outer cover and the inner cushion are integrated in one part. The outer cover is fitted with straps to attach the Allrounder to your body. The inner cushion is made up of zippered compartments. Each compartment is filled with a specific number of triangular, air filled SmartCells™.

Seat cover

The seat cover is made of water- and incontinence-resistant material, which is two-way stretchable. The seat cover is zipped to the outer cover.

INSTALLATION

Vicair recommends installation and adjustment of the Allrounder under the direct supervision of your therapist or dealer.

- Remove the clear plastic wrapper.
- Place the Allrounder on a hard surface or a chair. Transfer from your wheelchair to the Allrounder and make sure you are well positioned.
- Close the waist strap. The strap must fit on or just over the waistband of your trousers.
- Close and tighten the two leg straps (not too tight). The elastic parts will minimise skin and tissue compression.

Checking for “bottoming out”

Your tailbone and pelvic bones should be supported by the air cells within the cushion; they must not touch the bottom of the Allrounder. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

- Sit on the Allrounder for approximately 20 minutes.
- Whilst seated on the Allrounder, lean forward.
- Let your therapist or dealer slide a hand between the cushion and the buttocks.
- Make sure the buttocks are supported by at least one full layer of SmartCells.
- Do not use the Allrounder if bottoming out occurs. In most cases, adjusting the filling grade of the compartments will solve the problem.

Adjusting the cushion filling

If necessary, the firmness can be adjusted by removing or adding cells from the various compartments of the inner cushion.

USE

Checking for redness of the skin

Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, stop using the Allrounder. Consult your therapist or dealer.

Use at high altitude

Because each SmartCell™ is filled with air, the Allrounder may behave differently when used at high altitude or in an aircraft. Under such circumstances, the cushion's firmness can be adjusted by removing a few cells from each compartment. Vicair recommends not to use the Allrounder above 2400 m (8000 ft) ASL (above sea level).

Water

The Allrounder has been designed for both indoor and outdoor use. Although it has a water-resistant cover, the Allrounder is not waterproof. Do not take the Allrounder under water. If the Allrounder gets very wet during use (e.g. when sailing or canoeing), it must be allowed to dry completely before re-use.

Use during watersport

Vicair advise you to wear a life jacket when using an Allrounder during watersport activities.

Use in motor vehicles

Normal wheelchair cushions can render the vehicle's seatbelt system unreliable. The Allrounder can be used as a safe alternative if you require additional protection during transportation.

Transferring the Allrounder to another user

Before allowing a second person to use the Allrounder, clean the inner cushion and wash the seat cover. Disinfect both parts. Make sure the cushion is correctly adjusted to the second person.

Replacing the seat cover

Under normal use, Vicair recommends replacement of the seat cover every year. Replacement covers can be purchased separately.

Flammability

Do not use or leave the Allrounder close to a naked flame or a high temperature heat source.

MAINTENANCE AND CLEANING

 **Caution:** Do not steam clean or autoclave the Allrounder. The Allrounder cannot withstand excessive heat or pressure.

Checking for wear and broken cells

Every three months, check the Allrounder for wear and collapsed cells. In case of serious damage, or a large number of collapsed cells, stop using the Allrounder. Consult your dealer.

Cleaning the inner cushion

- Wipe the inner cushion with a damp cloth using a mild detergent. Difficult spots or stains can be cleaned by first saturating the spot and then using a soft brush to loosen the dirt.
- If it is necessary to remove the cells for cleaning, note the number of cells in each compartment.
- Do not use aggressive or abrasive cleaners, or sharp objects to clean the inner cushion.
- Do not immerse the inner cushion in water or any other liquid.
- Do not wash the filled inner cushion, or the cells, in a washing machine.

Cleaning the seat cover

The seat cover can be machine washed. See the instructions on the label attached to the inside of the cover.

Disinfecting the Allrounder

In case of soiling with body fluids, and when transferring the Allrounder to another user, the Allrounder must be disinfected to avoid cross contamination.

- Clean the inner cushion and the seat cover as described above. When washing the seat cover, preferably use an anti-bacterial agent.
- Spray disinfectant on the seat cover and the inner cushion. Bend the compartments outwards to make sure that the spray mist gets into the gaps.

WARRANTY

Vicair offers a limited warranty on defects in workmanship and materials for a period of two (2) years after the original date of purchase, provided the product has been used normally. Any damages arising from abnormal use, or caused by improper handling, cleaning or negligence are excluded from this warranty. Should a defect in materials or workmanship occur within two (2) years from the original date of purchase, Vicair will, at its own choice, repair or replace the product free of charge. The outer cover is excluded from this warranty.

Remedies for breach of express warranties herein are limited to repair or replacement of the product. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

For additional product information, see our website: www.vicair.com.

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