**NXT™ MULTIFIT HEAD SUPPORT**  
*SUPPORT DE TÊTE MULTIFIT NXT™*

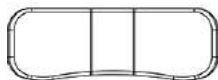
**INSTRUCTION MANUAL**  
*MANUEL D'INSTRUCTION*

# AVAILABLE COMPONENTS / COMPOSANTS DISPONIBLES

## PADS / COUSSINS

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**Extra Large Pad**  
*Coussin extra large*  
NIB-0818

**Large Pad**  
*Grand coussin*  
NIB-0784


**Contoured Pad**  
*Coussin profilé*  
NIB-0785

## HARDWARE / MATÉRIEL

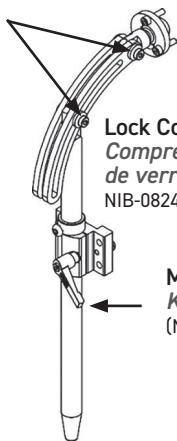**Includes two additional  
Quick Locking Handles**  
(for optional tool-free  
adjustment)

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**Standard / Standard** (NIB-0747)

**Flip Back / Rabattable** (NIB-1306)

**Universal Mount**  
(for third party Head Supports)

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# HEAD SUPPORT INSTALLATION

## INSTALLATION DESUPPORT DE TÊTE

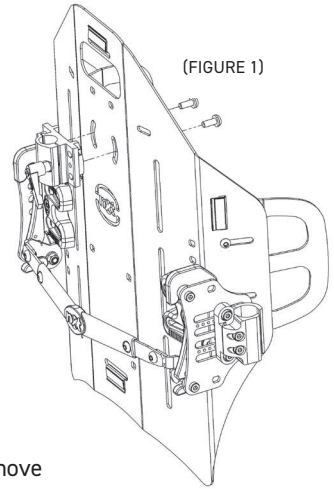
Tool Required: 4mm Allen Key / Outil requis: Clé Allen 4mm

### Step /Étape #1

#### Mounting Kit / Kit de montage (Figure 1)

Attach the mounting kit hardware to the back shell using the 2 screws and washers provided (also applies to Universal Mount)

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### Step /Étape #2

#### Head Support / Support de tête

The Head Support's post can be installed in one of three orientations - center, left, or right. To change the orientation, move the channel screw, located on the inside of the Mounting Bracket, to the appropriate location.

Next, slide the Head Support's vertical post through the mounting bracket and secure in place using the locking handle (Figure 3). The channel in the vertical post should slide over the Channel Screw (Figure 2), ensuring the post cannot rotate after installation.

Set the post to the desired height, and slide the locking collar down to the top of the mounting kit. Secure in place by tightening its screw against the head support vertical post. This will ensure that if the head support is removed, the height of the head support will always return to same height that it is set to when re-installed.

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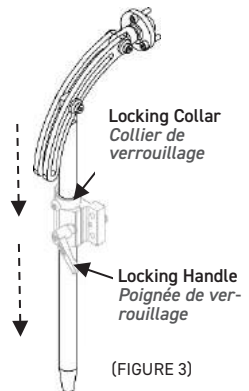
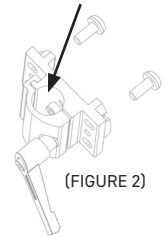
Channel screw used to adjust Head Support orientation

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
### Step /Étape #3

#### Head Support Pad / Coussin de support de tête

Unzip the back of the pad cover exposing the inside of the pad. Set the ball on the head support into the receiver inside the back of the pad. Secure using the 3 screws provided (Figure 4). The pad can then be bent in or out to customize the fit for the user.

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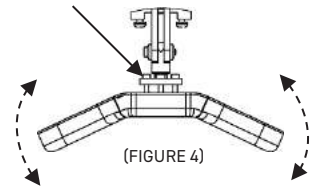
#### Optional Step / Étape facultative

##### Install Flip Back Hardware (if ordered)

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To install the optional flip back hardware, first remove the head support arm by unfastening the hardware (3 pieces) that secures it (Figure 5). Set the hardware and head support arm aside to reattach after the flip back has been installed.

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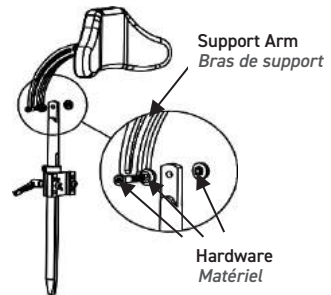
(FIGURE 5)

Next, install the flip back post extension over the existing post. With the hardware provided, secure the flip back post extension by inserting the locking handle through the channel (with thin washer and heavy duty washer) and tightening the special nut on the other side (Figure 6).

Note: both the heavy duty washer and the special nut must be installed with the small two sided protrusion going into the elongated holes on each side of the flip back post extension

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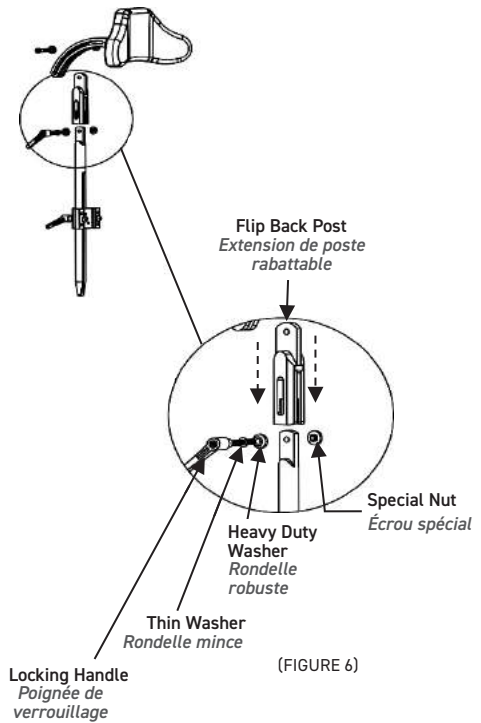
Last reinstall the head support arm over the flip back post extension, securing in place with the hardware that was set aside earlier in this step (Figure 7).

Note: The additional Locking Handles Assembly that is provided can be installed here instead of the original hardware if easier readjustment is desired.

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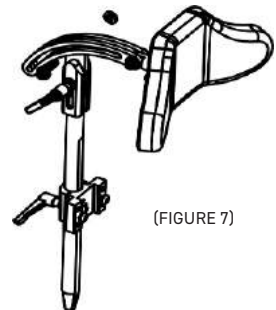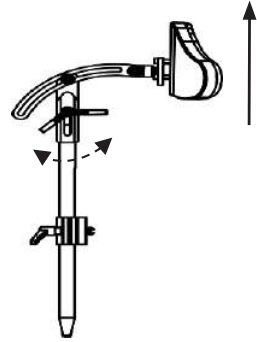
To loosen, turn the latch clamp between  $\frac{1}{2}$  - 1 full turn **maximum**. Raise the post extension as far as possible.

Note: Once the post extension has been raised, hold onto the support arm and head pad to prevent it from falling forward.

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### Step / Étape #2

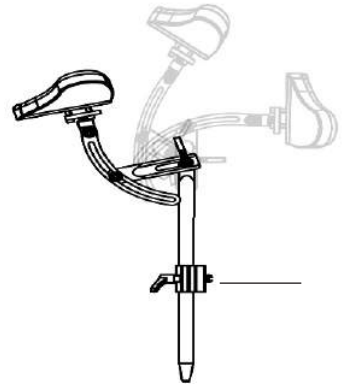
#### Flip Back / Rabattable

The head pad can now be flipped back, away from the user.

Note: Reverse the process to return the head support pad to its original position. Tighten the locking handle to prevent any movement. The locking handle can be repositioned on the screw once it is tightened by pulling out and turning.

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1422791 Ontario Ltd

Dynamic Health Care Solutions

753011 Second Line EHS, Orangeville, ON L9W 5W4 Canada

519-942-8441 | [www.DynamicHCS.com](http://www.DynamicHCS.com)

Advena Limited | Tower Business Centre, 2nd Flr., Tower Street Swatar, BKR 4013 Malta